



INGREDIENTS
Sodium chloride
Ultra-filtered water

0,9 %

DURABILITY
24 month. See best before date.

For external use only.
Reseal after use.

Saline

0.9% physiological sodium chloride

RADICIN Saline - is a 0.9% physiological sodium chloride solution suitable for cleaning the eyes, ulcers and skin lesions.

Washer fluid in the form of saline solution is used to flush away loose dirt from eg wounds or skin rashes. TRIKEM Saline comes in a plastic bottle with a cork tip. Pressure the bottle to get a strong directional beam with good flushing action. Slight pressure on the bottle also gives good precision when flushing the eyes.

Usage

Turn top of tip to open.

At wound cleaning - align the spout to the wound or injury and squeeze vigorously the bottle to get high coil power.

Eye cleaning - squeeze gently on the bottle and be careful not to damage the spout eye.

Reseal with the cap after use.

Can be stored and used even after the seal clamped.

Product Range

Art.no	EAN	Size	Pack
1852020	7394244102693	200 ml	12



Trikem AB
Ridspölgatan 11
SE213 77 Malmö, Sweden
Tel. +46 40 94 40 10 www.trikem.se